Cervical Sizer

for use with the RUMI® Koh-Efficient™ or KOH Colpotomizer™ System
KC-SIZER

Directions for Use (English)

**NON-STERILE
STERILIZE BEFORE USE**

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

DEVICE DESCRIPTION

The Cervical Sizer (Sizer) is a medical device used to determine the diameter of a cervix and for correctly sizing a colpotomy cup.

Not made with natural rubber latex.

CAUTION

U.S. Federal law restricts this device to sale by or on the order of a physician.

INDICATIONS FOR USE


The Sizer is designed for use in conjunction with the Koh-Efficient™ or KOH Colpotomizer™ System, where the cervix is present.

CONTRAINDICATIONS

The absence of a cervix.

PRECAUTIONS

- Do not use if the measuring platforms on the distal end of the device are not reading properly or are damaged.
- Sterilize before use.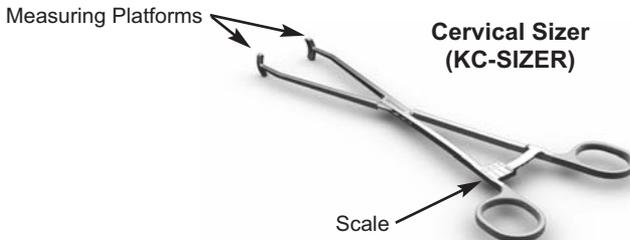

DIRECTIONS FOR USE

Calibration test prior to insertion: To confirm that the Sizer is within its calibrated parameter, set the Scale on the Sizer to the 4 cm position and hold it up to a 4 cm KOH Cup™ or metric scale. If the device is not calibrated correctly, do not use.

1. Under direct visualization, place the Sizer around the cervix at the fornix in the 4.0 position (Figure 1).
2. Close until you approximate the cervix (Figure 2). **Caution: Compression of the cervix may cause a false small measurement.**
3. Take note of the cup size indicated on the Scale of the Sizer (Figure 3).
4. Once the correct cup size is determined, remove the Sizer.

Figure 1

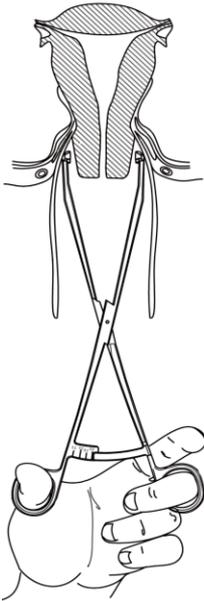
Figure 2


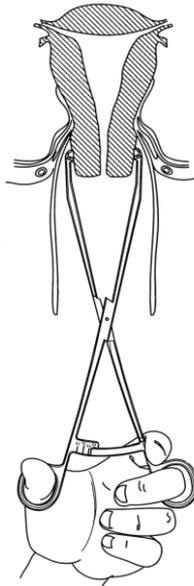
Figure 3

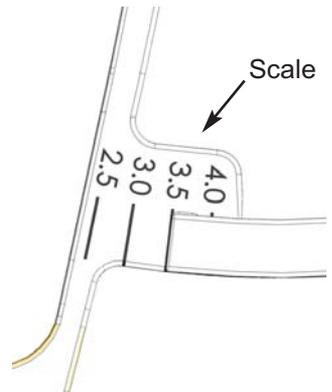RECOMMENDED CLEANING INSTRUCTIONS

Always rinse and clean instrument immediately following use. Enzymatic cleaning agents reduce organic matter and facilitate its removal. Never use corrosive cleaning agents (i.e. bleach). Neutral pH (7.0) cleaning agents are recommended (if unknown, check MSDS or check with distributor).

1. Prepare the enzymatic detergent according to the manufacturer's recommendations and submerge the Sizer in the detergent. Soak the Sizer for ten (10) minutes or as directed.
2. Scrub the submerged Sizer with a soft sponge and agitate. Clean crevices, using a pipe cleaner/brush or flush with a syringe when appropriate. Actuate any moving parts to loosen trapped soil.
3. Rinse in warm 100 °F – 120 °F (38 °C – 49 °C) tap water for one (1) minute. Thoroughly flush. Actuate while rinsing.
4. Ultrasonically clean the Sizer for ten (10) minutes in a neutral pH detergent (Neutrad® or acceptable alternative). Prepare the detergent according to the manufacturer's recommendations.
5. Rinse the Sizer with clean tap water, actuating moving parts while rinsing for one (1) minute, repeat rinse twice.
6. Dry the Sizer thoroughly with a clean, lint-free cloth.
7. Repeat this cleaning procedure if the Sizer appears to be soiled after initial cleaning until there is no sign of soil on the device.

STORAGE

The Cervical Sizer must be stored dry in a moisture free area. The device should be stored individually in a protective tray with partitions, if possible. Protect with cloth or gauze if stored in drawers.

RECOMMENDED STEAM AUTOCLAVE STERILIZATION PARAMETERS

- The Sizer must be thoroughly cleaned of all foreign matter prior to sterilization.
- Follow the manufacturer's instructions for operation and loading of steam autoclave.
- There must be direct steam exposure to all surfaces of the device being sterilized.
- Allow Sizer to cool down from autoclave to room temperature. Do not immerse in any fluid until the device has been allowed to cool.

Pre-vacuum Cycle

Preconditioning Pulses: 4

Exposure temperature: 270 °F – 273 °F (132 °C – 134 °C)

Exposure time: 4 minutes

Gravity Displacement Cycles

Exposure temperature: 270 °F – 275 °F (132 °C – 135 °C)

Exposure time: 15 minutes

Exposure temperature: 250 °F – 253 °F (121 °C – 123 °C)

Exposure time: 40 minutes

WARRANTY

The Cervical Sizer is warranted by CooperSurgical for a period of one (1) year from date of shipment. This warranty applies only to the original purchaser and only against defects from workmanship or materials, which under normal use render the device inoperable.

EXPLANATION OF SYMBOLS

REF Reorder Number

LOT Batch Code

R_x Only U.S. Federal law restricts this device to sale by or on the order of a physician.

 Consult instructions for use

CE Product conforms to the Medical Device Directive 93/42/EEC

EC REP Authorized Representative in the European Community.

 Manufacturer

Not made with natural rubber latex.

Neutrad® is a registered trademark of Decon Labs, Inc.
All other marks bearing the symbols “TM” and “®” are trademarks and registered trademarks of CooperSurgical, Inc.

© 2014 CooperSurgical, Inc. All rights reserved.

95 Corporate Drive • Trumbull, CT 06611 USA
Phone: (800) 243-2974 • Fax: (800) 262-0105
www.coopersurgical.com

International
Phone: (203) 601-9818
Fax: (203) 601-4747

CooperSurgical, Inc.
95 Corporate Drive
Trumbull, CT 06611

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© 2014 CooperSurgical, Inc. Todos os direitos reservados.