

# INSTRUCTIONS FOR USE

## DESCRIPTION:

Oleeva Clear and Oleeva Fabric are silicone adhesive sheets used to reduce or prevent hypertrophic and keloid scars.

## USE:

Consistent use of Oleeva can minimize or assist in the prevention of existing hypertrophic and keloid scars.

## INSTRUCTIONS:

- **STEP 1:** Remove product from packaging. For full coverage, we recommend that you do not alter the size of the product; If necessary you can cut the excess product with scissors.
- **STEP 2:** Remove the protective paper and save it for later use. Place the sticky part of the product down, directly on the scar. The area of the scar should be completely healed before placing the product. You can use the sheet or patch of Oleeva for at least 12 hours and up to 23 hours a day.
- **STEP 3:** Wash the area of the scar at least once a day. You should also wash the patch once a day with warm water and mild soap. Do not use soaps that have lotions or oils. You can use Bio Med soap, Silicone Gel Cleanser. Rinse the patch to remove any trace of soap. Allow the product to air dry or apply Oleeva to your skin even if it is wet, but dry the back with a cloth or towel that does not leave residue stuck in the patch. Between uses, keep the patch on the protective paper on which it came.



## RECOMMENDATIONS AND PRECAUTIONS:

Do not use creams, lotions or ointments with this product because it may interfere with its effectiveness or cause irritation of your skin.

## CAUTION:

Like any other topical product applied directly to the skin, some type of reaction (usually irritation) may occur. Stop using the product if you have any reaction. When your skin has returned to its normal state, you can begin to reapply Oleeva for short periods of time and gradually increase hours of use. Look at the area for reaction signs. Continue increasing the usage time until it reaches the recommended maximum. If you continue to have problems, stop using it and consult your physician.

## **WARNING!**

DO NOT USE IN OPEN WOUNDS.



[www.PelegrinaMedical.com](http://www.PelegrinaMedical.com) / [www.PelegrinaMedical.net](http://www.PelegrinaMedical.net)

Tel. +1.787.761.0000 Fax. +1.787.761.0404

Email. [Sales@pelegrina.com](mailto:Sales@pelegrina.com)