

GastroFlush®

Bionix® GastroFlush® with ENFit® connection helps providers safely clear feeding tube obstructions and avoid costly emergency feeding tube replacement.



PROBLEM

Clogged feeding tubes can result in preventing delivery of medication and nutrition, patient discomfort, and increased costs such as transportation, extended stay and procedures due to complications.

SOLUTION

GastroFlush, an easy to use handheld device, uses pulsating water to quickly unclog a feeding tube at the bedside.

RESULTS

Feeding tube patency restored in a safe, efficient, cost effective manner.

PRODUCT OFFERING

#9510 - GastroFlush™ (10/box)

#9500 - Relish RL1501 Jet Lavage



SINGLE
USE ONLY

R_x ONLY

MADE IN
THE U.S.A.



INNOVATIVE SOLUTIONS FOR EVERYDAY PROBLEMS

800.551.7096 • www.bionix.com

©2019 Bionix Development Corp. | MKT-0095 | Rev. B


GastroFlush®

EFFECTIVELY AND SAFELY REMOVE ENTERAL FEEDING TUBE OCCLUSIONS

GastroFlush has been tested for safety and reliability.

Bench Test Conducted	Test Type	Tube Tested	Number of Tests	Test Results	CR Level
Clog Removal	Simulated Use	8Fr-24Fr, 42" (max)	n=30/tube size	8Fr = 100% Success 24Fr = 100% Success	95/90
Time to Remove Clog	Simulated Use	8Fr-24Fr, 42" (max)	n=30/tube size	8Fr = 2.20 seconds (AVG) 24Fr = 1.79 seconds (AVG)	
Pressure Relief Valve	Valve Opening	8Fr-24Fr, 42" (max)	n=15 assemblies	AVG + SD < 15 PSI	
Leak Free Pressure	Applied Pressure	8Fr-24Fr, 42" (max)	n=30/tube size	100% Leak Free up to 30 PSI	

*Laboratory Benchtop testing was conducted at Bionix under worst case scenarios. Tubes were occluded using a coagulated protein and fiber occlusion. GastroFlush™ user instructions were used to prepare and clear occlusions. Stopwatch was used to measure time from activating the jet lavage to visible occlusion removal (checked with syringe).

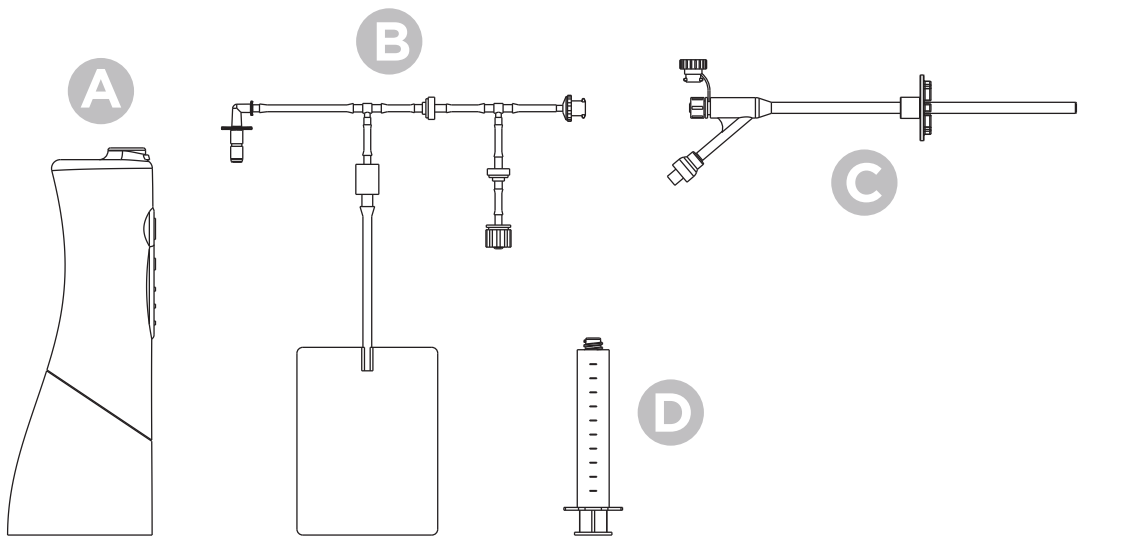

**(A) GF-09500
Relish RL1501 Jet Lavage**

- Accessory
- Sold Separately
- Needed to operate GastroFlush

**(B) GF-09510
GastroFlush™**

- 10/box
- Disposable Assembly
- Only for use with Relish RL1501 Jet Lavage

(D) Enfit® Syringe

- 5ml
- Included with purchase of GastroFlush

(C) Enfit® Feeding Tube

- *In dwelling - not provided with purchase

*GastroFlush is for enteral feeding tubes with integral ENFit connectors only. It is **NOT** to be used with Drain Tubes or ENFit transition adaptors.

US Patent # 10,258,543. ENFit® and NRFit™ are trademarks of GEDSA.



INNOVATIVE SOLUTIONS FOR EVERYDAY PROBLEMS

800.551.7096 • www.bionix.com