

Milex® Pessaries

Making Life Easier for You and Your Patient

CooperSurgical offers:

- Office inventory management – ensure most commonly used pessaries and quantities in stock
- Superior knowledge of your account – reduces over-stock issues
- Experts in specific indications for which the different pessaries are utilized
- In-service/re-training on the different indications and patient profiles
- Excellent product quality and consistency
- Superior Customer Service
- Patient education materials



Milex® Pessaries Fitting Kit.....57

Milex® Pessaries for
Prolapse and Incontinence58

Milex® Pessaries Sizing Charts.....62

Milex® Pessaries

Our Pessaries Are Autoclavable

Milex Pessaries are used to help patients with pelvic organ prolapse (POP) and urinary incontinence. They come in 25 different styles and many sizes to fit each patient's anatomy. Made of soft, autoclavable, medical-grade silicone, Milex Pessaries are an excellent addition to any practice that treats women with lower urinary tract dysfunction.

Milex® Pessary Fitting Kit



The Milex Pessary Fitting Kit was developed to help you in your practice to determine accurate sizing prior to prescribing a pessary for your patient. Note: for fitting only, not for long-term patient use.

| Number | Description |
|----------|---|
| MXFIT | Pessary Fitting Set (For sale in the USA only) |
| MXFIT-CO | Pessary Fitting Set Case and Tray only (no pessaries) |
| MXFIT-CH | Pessary Fitting Set (China) Note: Limited selection of Gellhorns, Rings with support and Incontinence Dish without Trimo-San |
| MXFIT-CA | Pessary Fitting Set (International) Note: As seen in above photo but without Trimo-San |

ACCESSORIES

CA* CE

| Number | Description | Number | Description |
|-----------|--|-----------|--------------------------------------|
| MXFIT0016 | Fitting Gellhorn Short Stem Flexible 2¼" | MXFIT0801 | Fitting Incont. Dish with support #2 |
| MXFIT0017 | Fitting Gellhorn Short Stem Flexible 2½" | MXFIT0802 | Fitting Incont. Dish with support #3 |
| MXFIT0018 | Fitting Gellhorn Short Stem Flexible 2¾" | MXFIT0803 | Fitting Incont. Dish with support #4 |
| MXFIT0019 | Fitting Gellhorn Short Stem Flexible 3" | MXFIT0804 | Fitting Incont. Dish with support #5 |
| MXFIT0012 | Fitting Gellhorn Flexible 2¼" | MXFIT0805 | Fitting Donut #2 |
| MXFIT0013 | Fitting Gellhorn Flexible 2½" | MXFIT0806 | Fitting Donut #3 |
| MXFIT0014 | Fitting Gellhorn Flexible 2¾" | MXFIT0807 | Fitting Donut #4 |
| MXFIT0015 | Fitting Gellhorn Flexible 3" | MXFIT0808 | Fitting Donut #5 |
| MXFIT0020 | Fitting Ring with support #2 | MXFIT0809 | Fitting Cube #2 |
| MXFIT0021 | Fitting Ring with support #3 | MXFIT0810 | Fitting Cube #3 |
| MXFIT0022 | Fitting Ring with support #4 | MXFIT0811 | Fitting Cube #4 |
| MXFIT0023 | Fitting Ring with support #5 | MXFIT0812 | Fitting Cube #5 |
| MXFIT0024 | Fitting Ring with support #6 | MXFIT0072 | Manikin |
| MXFIT0025 | Fitting Ring with support #7 | | |

Trimo-San™ – Vaginal Jelly

Trimo-San helps restore and maintain the normal vaginal acidity and coats the walls of the vagina with a lubricating film that helps reduce odor-causing bacteria. Trimo-San is a recommended adjunct for pessary wearers. **For sale in the USA only.**



| Number | Description |
|--------|---|
| MX5030 | Trimo-San Cream with Jector (MXAPP Jel-Jector replacement at no charge) |

Pessary PowerPoint CD

CA* CE

A detailed educational presentation for medical professionals on pessary use in a Microsoft PowerPoint format. An excellent tool when making a presentation on pessaries and their uses for many gynecologic indications. Also includes information on the history of the pessary and on insurance reimbursement.



| Number | Description |
|--------|-----------------------|
| MPESCD | Pessary PowerPoint CD |

To place an order, call us at: 800.243.2974
Visit us online at: www.CooperSurgical.com

Milex® Pessaries for Prolapse and Incontinence (See pages 62-63 for sizing chart)

P Prolapse **I** Incontinence


Ring / Folding **P**



For mild prolapse

Number

MXKPER + Size **CA** **CE**

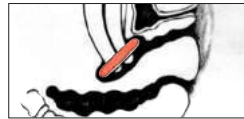

Ring with Knob / Folding **P** **I**



For mild prolapse complicated by stress urinary incontinence

Number

MXKPERK + Size **CA** **CE**



Ring with Support / Folding **P**



For mild prolapse complicated by a mild cystocele

Number

MXKPRS + Size **CA** **CE**



Incontinence Ring / Flexible **I**



For stress urinary incontinence

Number

MXKPCON + Size **CA** **CE**




Ring with Support and Knob / Folding **P** **I**



For mild prolapse complicated by a mild cystocele and stress urinary incontinence

Number

MXKPRSK + Size **CA** **CE**

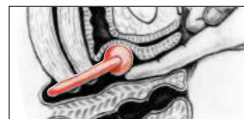

Shatz / Folding **P**



For mild prolapse complicated by mild cystocele

Number

MXKPSH + Size **CA** **CE**



P Prolapse **I** Incontinence

Donut

P



Support for third-degree prolapse/proctientia. Not to be deflated during insertion or removal. See Inflatoball if a deflatable pessary is necessary

Number

MXKPDO + Size **CA** **CE**



Gellhorn / Flexible – Short Stem

P



For third-degree prolapse/proctientia. Length is 1-inch from base to tip

Number

MXKPGSS + Size **CA** **CE**



Regula / Folding

P



For uterine prolapse (first and second degree)

Number

MXKPREG + Size **CA** **CE**



Gellhorn / Flexible

P



For third-degree prolapse/proctientia. Length is 1½-inch from base to tip

Number

MXKPGE + Size **CA** **CE**



Hodge with Support and Knob / Folding

I



For stress urinary incontinence complicated by a mild cystocele in the woman who has a small vaginal introitus

Number

MXKPHSK + Size **CA** **CE**



Hodge with Knob / Folding

I



For stress urinary incontinence in the woman who has a small vaginal introitus

Number

MXKPEHK + Size **CA** **CE**



Milex® Pessaries for Prolapse and Incontinence (See pages 62-63 for sizing chart)

P Prolapse **I** Incontinence

Hodge with Support / Folding **I**



For stress urinary incontinence with a mild cystocele for a woman who has a very small vaginal introitus

Number

MXKPHS + Size **CA** **CE**



Hodge / Folding **I**



For stress urinary incontinence in a woman who has a very small vaginal introitus. May be helpful in cases of repeated miscarriages due to an incompetent cervix by repositioning the weight on the growing fetus

Number

MXKPEH + Size **CA** **CE**




Incontinence Dish / Folding **P** **I**




For stress urinary incontinence with mild prolapse

Number

MXKPCOND + Size **CA** **CE**

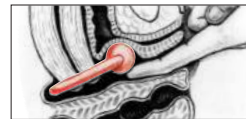

Risser / Folding **I**

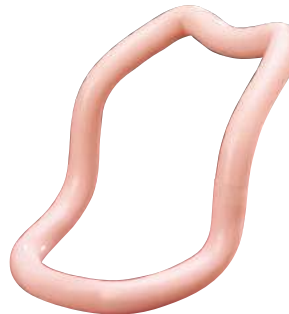

Designed for the patient with a shallow pubic notch. Has larger weight-bearing zone to support vaginal aspects of pubis. May be helpful in cases of repeated miscarriages due to an incompetent cervix by repositioning the weight of the growing fetus

Number

MXKPRI + Size **CA** **CE**



Incontinence Dish with Support / Folding **P** **I**



For stress urinary incontinence with mild prolapse and mild cystocele

Number

MXKPCONDS + Size **CA** **CE**



Smith / Folding **I**



Modification of Hodge Pessary. For stress urinary incontinence, designed for patients with a well-defined pubic notch. May be helpful in cases of repeated miscarriages due to incompetent cervix by repositioning the weight of the growing fetus

Number

MXKPES + Size **CA** **CE**



P Prolapse **I** Incontinence

Gehrung with Knob / Folding **I**



For cystocele with or without mild rectocele complicated by stress urinary incontinence

Number

MXKPGSK + Size **CA** **CE**



Gehrung / Folding **P**




Support for cystocele and rectocele. It is also more adaptable in cases of procidentia where the uterus tends to herniate when other pessaries are used

Number

MXKPGS + Size **CA** **CE**



Cube **P**

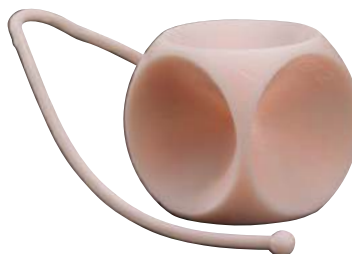

Support for third-degree prolapse/procidentia. Should be removed nightly

Number

MXKPEC + Size **CA** **CE**

Cube with drainage holes also available

Number

MXKPECH + Size **CA** **CE**



Tandem-Cube **P**



Support for third-degree prolapse/procidentia when single cube (above) provides inadequate support. Should be removed nightly

Number

MXKPECT + Size **CA** **CE**



Millex® Inflatoball **P**



Inflatable for third-degree prolapse/procidentia with or without a mild cystocele and/or rectocele. Should be removed nightly

Number

MXKPINF + Size

CA **CE**




100% Latex

Milex® Pessary Sizing Chart

Shaded areas show the minimum assortment necessary to fit approximately 85 percent of patients.

Hodge — MXKPEH


CA* CE

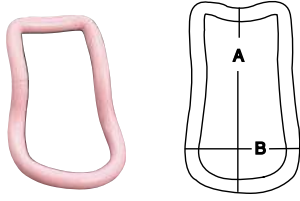

| Size | in. (A) | in. (B) | mm (A) | mm (B) |
|------|---------|---------|--------|--------|
| 0 | 2½" | 1¼" | 70 | 44 |
| 1 | 3" | 1⅝" | 76 | 48 |
| 2 | 3¼" | 1⅞" | 83 | 49 |
| 3 | 3½" | 2" | 89 | 51 |
| 4 | 3¾" | 2¼" | 92 | 54 |
| 5 | 3⅞" | 2½" | 95 | 57 |
| 6 | 4" | 2⅝" | 98 | 60 |
| 7 | 4¼" | 2¾" | 108 | 64 |
| 8 | 4½" | 2⅞" | 117 | 67 |
| 9 | 5" | 3" | 127 | 70 |

Smith — MXKPES


CA* CE

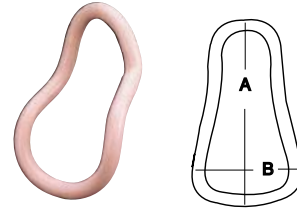

| Size | in. (A) | in. (B) | mm (A) | mm (B) |
|------|---------|---------|--------|--------|
| 0 | 3⅝" | 2" | 79 | 51 |
| 1 | 3¾" | 2⅜" | 83 | 54 |
| 2 | 3⅞" | 2½" | 89 | 57 |
| 3 | 4" | 2⅞" | 95 | 60 |
| 4 | 4¼" | 3" | 108 | 64 |
| 5 | 4½" | 3¼" | 114 | 67 |
| 6 | 4¾" | 3½" | 121 | 70 |
| 7 | 5" | 3¾" | 127 | 73 |
| 8 | 5½" | 4" | 140 | 76 |
| 9 | 5¾" | 4¼" | 146 | 79 |

Gehrung — MXKPGS


CA* CE

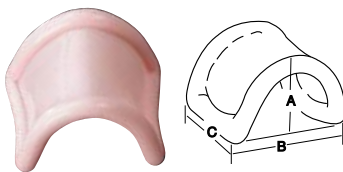

| Size | in. (A) | in. (B) | in. (C) | mm (A) | mm (B) | mm (C) |
|------|---------|---------|---------|--------|--------|--------|
| 0 | 1⅙" | 1½" | 1⅜" | 27 | 38 | 35 |
| 1 | 1⅛" | 1¾" | 1½" | 29 | 44 | 38 |
| 2 | 1⅞" | 2" | 1⅞" | 35 | 51 | 41 |
| 3 | 1½" | 2⅞" | 1¾" | 38 | 54 | 44 |
| 4 | 1⅞" | 2¼" | 1⅞" | 41 | 57 | 48 |
| 5 | 1¾" | 2⅞" | 2" | 44 | 60 | 51 |
| 6 | 1⅞" | 2½" | 2⅞" | 48 | 64 | 54 |
| 7 | 2" | 2⅞" | 2¼" | 51 | 73 | 57 |
| 8 | 2⅞" | 3" | 2⅞" | 54 | 76 | 60 |
| 9 | 2¼" | 3⅞" | 2½" | 57 | 79 | 64 |

Regula — MXKPREG

CA* CE

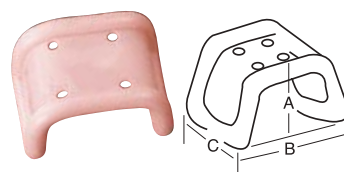

| Size | in. (A) | in. (B) | in. (C) | mm (A) | mm (B) | mm (C) |
|------|---------|---------|---------|--------|--------|--------|
| 0 | 1⅙" | 1½" | 1⅜" | 27 | 38 | 35 |
| 0 | 1" | 1⅞" | 1⅜" | 25 | 49 | 35 |
| 1 | 1⅙" | 2" | 1½" | 27 | 51 | 38 |
| 2 | 1" | 2⅞" | 1⅞" | 29 | 54 | 51 |
| 3 | 1¼" | 2⅞" | 1¾" | 32 | 56 | 44 |
| 4 | 1⅞" | 2¼" | 1⅞" | 35 | 57 | 48 |
| 5 | 1⅞" | 2⅞" | 2" | 37 | 60 | 51 |
| 6 | 1½" | 2⅞" | 2⅞" | 38 | 67 | 54 |
| 7 | 1⅞" | 2⅞" | 2¼" | 41 | 73 | 57 |
| 8 | 1¾" | 3⅞" | 2⅞" | 44 | 79 | 60 |

Ring with Support — MXKPRS CA* CE



| Size | in. | mm |
|------|-----|-----|
| 0 | 1½" | 44 |
| 1 | 2" | 51 |
| 2 | 2¼" | 57 |
| 3 | 2½" | 64 |
| 4 | 2¾" | 70 |
| 5 | 3" | 76 |
| 6 | 3¼" | 83 |
| 7 | 3½" | 89 |
| 8 | 3¾" | 95 |
| 9 | 4" | 102 |
| 10 | 4¼" | 108 |
| 11 | 4½" | 114 |
| 12 | 4¾" | 121 |
| 13 | 5" | 127 |

Incontinence Ring — MXKPCON CA* CE



| Size | in. | mm |
|------|-----|-----|
| 0 | 1½" | 44 |
| 1 | 2" | 51 |
| 2 | 2¼" | 57 |
| 3 | 2½" | 64 |
| 4 | 2¾" | 70 |
| 5 | 3" | 76 |
| 6 | 3¼" | 83 |
| 7 | 3½" | 89 |
| 8 | 3¾" | 95 |
| 9 | 4" | 102 |
| 10 | 4¼" | 108 |
| 11 | 4½" | 114 |
| 12 | 4¾" | 121 |
| 13 | 5" | 127 |

Shaatz — MXKPSH CA* CE



| in. | mm | in. | mm |
|-----|----|-----|----|
| 1½" | 38 | 2¾" | 70 |
| 1¾" | 44 | 3" | 76 |
| 2" | 51 | 3¼" | 83 |
| 2¼" | 57 | 3½" | 89 |
| 2½" | 64 | 3¾" | 95 |

Cube — MXKPEC CA* CE



| Size | in. | mm |
|------|-----|----|
| 0 | 1" | 25 |
| 1 | 1⅙" | 30 |
| 2 | 1⅛" | 35 |
| 3 | 1½" | 38 |
| 4 | 1⅞" | 41 |
| 5 | 1¾" | 44 |
| 6 | 2" | 51 |
| 7 | 2¼" | 57 |

Donut — MXKPDO CA* CE



| in. | mm | in. | mm |
|-----|----|-----|----|
| 2" | 51 | 3" | 76 |
| 2¼" | 57 | 3¼" | 83 |
| 2½" | 64 | 3½" | 89 |
| 2¾" | 70 | 3¾" | 95 |

Incontinence Dish — MXKPCOND CA* CE



| Size mm | Size mm |
|---------|---------|
| 55 | 75 |
| 60 | 80 |
| 65 | 85 |
| 70 | |

Gellhorn — MXKPGE CA* CE



| in. | mm | in. | mm |
|-----|----|-----|----|
| 1½" | 38 | 2¾" | 70 |
| 1¾" | 44 | 3" | 76 |
| 2" | 51 | 3¼" | 83 |
| 2¼" | 57 | 3½" | 89 |
| 2½" | 64 | 3¾" | 95 |

Inflatoball — MXKPINF CA* CE



| Size | in. | mm |
|------|-----|----|
| S | 2" | 51 |
| M | 2¼" | 57 |
| L | 2½" | 64 |
| XL | 2¾" | 70 |

Premier Products for Women's Health Care

heroption®
in-office cryoablation therapy

LEEP PRECISION™

EURO-MED®

H/S Elliptosphere™

MILEX®

frigitronics®

LEISEGANG®

PIPELLE®

SPERMSELECT®



CooperSurgical

95 Corporate Drive
Trumbull, CT 06611