


**nio**  
A D U L T

**S.U.P.S.**




This is a two-handed procedure. Applying firm downward pressure allows for proper deployment.

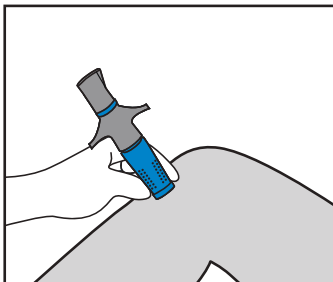


### STABILIZE

Stabilize the limb before the IO procedure. With your non-dominant hand, hold the NIO at the base by the textured dots, at a 90° angle to the surface against the skin. Holding it like a dart, fan out your other fingers to **stabilize** the NIO against the skin.

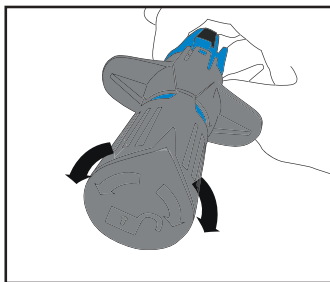


### UNLOCK

**Unlock** the NIO by rotating the cap 90° in either direction.

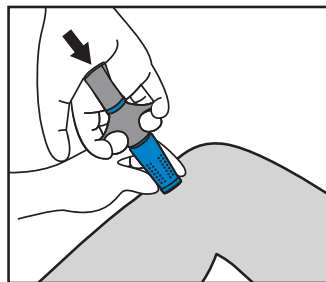

### PRESS

Place the palm of your dominant hand over the cap of the NIO and **apply downward pressure**, while your non-dominant hand stabilizes the NIO against the patient's skin.

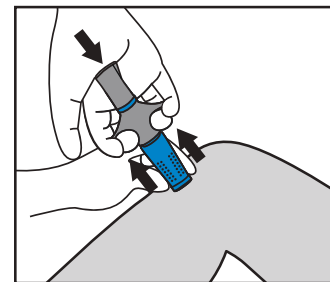

### SQUEEZE

With your dominant hand, maintain downward pressure on the cap and **squeeze** the trigger wings. Make sure device and limb are stable during device deployment.



by SAFEGUARDMEDICAL™

visit [safeguardmedical.com](http://safeguardmedical.com) • call +1 (855) 428-6074 • email [info@safeguardmedical.com](mailto:info@safeguardmedical.com)

301700684\_Rev.4\_06-14-2023